
ZICO®

QUIC-RELEASE® MULTIVERSAL BRACKET MODEL QR-MV

3030PM1


REV 5-16-2024

To properly store your cylinder use all 4 straps.

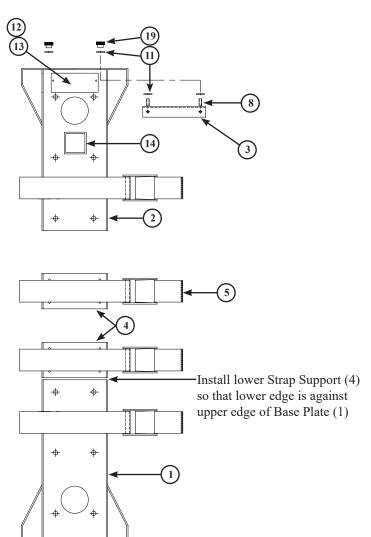

IMPORTANT:

Running end of strap "A" goes underlatch and back through front of latch between retainer and thumb release.

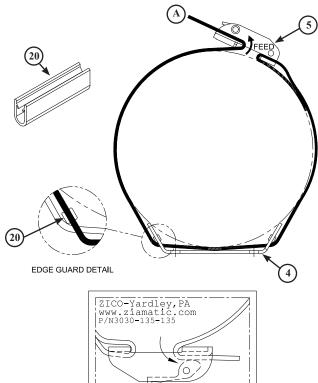

Strap threading label sewn on strap, near buckle

ITEM NO.	DESCRIPTION	PART NUMBER	PIECES REQUIRED
1	Base Plate Assembly	3030-135-105	1
2	Retainer Assembly	3030-135-110	1
3	Stop	3030-000-109	1
4	Support Strap	3030-135-120	2
5	Strap & Buckle Assembly	3030-135-135	4
8	1/4" x 3/4" Hex Head Bolt	9010-102512	2
11	1/4" Flat Washer	9014-112500	4
12	#4 Drive Screw	9020-101103	2
13	Label "CAUTION/MODEL NO."	3030-135-140	1
14	Label "KKK-A-1822-F"	3030-000-115	1
19	1/4-20 x 3/4" Dia. Thumbnut	9012-352500	2
20	Edge Guard, 2"	3030-135-145	8

This bracket has been tested to meet SAE-J3043, AMD Standard 003 and GSA KKK-A-1822E. This should be taken into consideration when selecting the mounting location.


Mount using 5/16" Grade 8 or better hardware with washers.

Straps should be inspected often and replaced if they show signs of wear or fraying. Otherwise, they should be replaced every three(3) years to ensure adequate retension.

The tag below is mounted to the retainer bracket(1) and marked with the replacement date.



3030-135-140 Model/Caution Tag with replacement date in the bottom right corner.

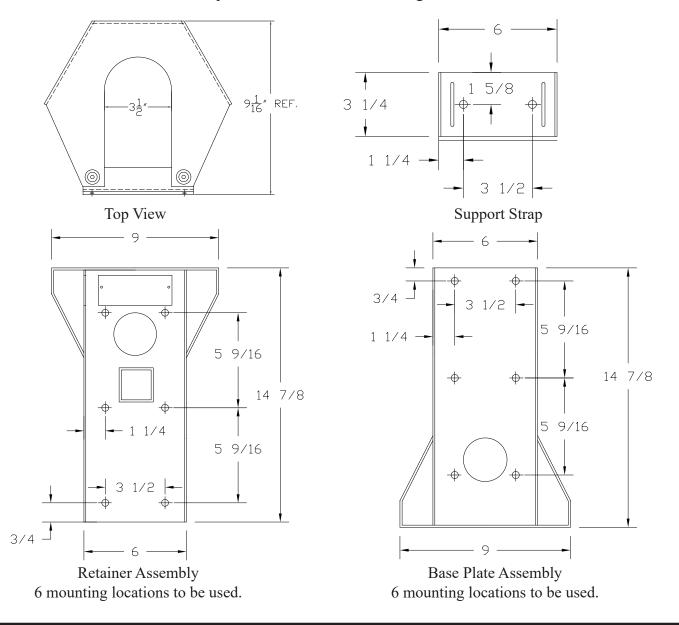



www.ziamatic.com

TOLL FREE: 800-711-3473